

Animas Pump Training Checklist

Training Details

Patient Name: _____ Pump Trainer's Name: _____ Animas Pump Model: _____
 Date of Birth: _____ Location: _____ Serial Number: _____
 Training Date: _____ Saline Insulin
 Loan Own pump Upgrade

Training Number:

1. Patient Self-Study

- Reviewed User Guide
- Viewed Training DVD
- Completed Patient Workbook
- Completed Online Quiz

2. HCP to Complete

- Insulin for pumps (Rapid Acting)
- Carbohydrate counting
- Insulin to Carb Ratio (I:C)
- Insulin Sensitivity Factor (ISF)
- Correction issues and factors
- Review Target BG and range
- Review BG testing frequency
- Review symptoms, testing & treatment of Hypoglycaemia
- Review Glucagon Instructions
- Review Sick Day mgmt plan
- Review Ketone testing & preventing DKA
- Infusion set options
- Site selection, rotation and care
- Site preparation & set insertion
- Frequency & indications for site change
- Cannula fill amounts
- Lifestyle issues/exercising with the pump

3. Pump Trainer to Complete Pump overview and Programing

- Prepare Cartridge
- Prime/Rewind Menu
- Tightening cartridge cap
- Button functions, vents, battery cap, cartridge cap, IR window

- Battery Types, inserting battery & tightening cap
- Home screen
- Main menu screen
- Set date, time and sounds
- Advanced setup menu
- History feature
- Status screen
- Suspend/Resume
- Review/edit basal menu
- Setting temporary basal rates
- Multiple basal program options

Bolus Program

- Practice bolus menu options
 - Normal
 - ezCarb
 - ezBG
 - Combo Bolus
 - Food List
 - Reminders
- Cancelling a bolus
- Audio Bolus & ezBolus
- BG targets & Insulin on Board

Safety Systems & Alarms

- Disconnect procedure during Prime/Rewind menu and when tightening cartridge cap and replacing battery
- Locking / Unlocking the pump
- Auto Off
- Alerts, Alarms & Warnings
- Insulin limits
- Avoiding hazards
- X-Ray / CT Scans / MRI
- Free-fall rides / Roller coasters

Diasend Software

- Patient registered Patient Profile
- Downloading Data
- Accessing the Diasend Website
- Saving Pump settings
- Saving and Sending reports

Miscellaneous

- Wearing the pump
- Appropriate disposal of sharps etc
- Lifestyle issues / wearing the pump / exercising on the pump
- Appropriate disposal of supplies & sharps
- Travel Supplies to carry
- Ordering Supplies (NDSS)
- Ordering Accessories (AMSL)
- Pump Support Helpline
- When to call your Health Care Provider or Emergency Services
- Other: _____

I certify that the above ticked items are a true and accurate representation of the training given on the date listed above.

Patient/Guardian signature _____

Pump Trainer signature _____

CDE signature _____

Insulin Pumps are medical aid devices intended for use by patients requiring insulin infusion. Insulin Pumps are not a substitute for, and should not replace, regular communication and training between the patient and the diabetes health care team. Patient compliance is required for correct pump functionality. All pump starts must be performed by a health care professional with appropriate Animas training. It is the patient's responsibility to seek appropriate advice when necessary and to use the device as per usage instructions and warranty conditions. Please visit the AMSL Diabetes website for further details. www.amsldiabetes.com.au