

# SHARE YOUR CGM DATA WITH YOUR CLINIC

USE DEXCOM CLARITY® TO  
UNDERSTAND GLUCOSE PATTERNS AND  
ANALYSE OR SHARE DATA WITH YOUR CLINIC

- 1 Receive an invitation.  
Your clinic will print or email an invitation to you
- 2 Click <https://clarity.dexcom.eu/share> or enter it into your internet browser.  
Confirm to share before the expiration date
- 3 Click **Log in**.  
If you use G5 Mobile, you already have an account. Log in with your G5 Mobile or any Dexcom username and password.
- 4 Enter your username and password, then click **Log In**.
- 5 Enter the Temporary Authorisation Key from your invitation.
- 6 Enter your date of birth.
- 7 Click **Continue**.
- 8 Click **I consent to share my data with my clinic**.
- 9 Click **Yes, Share My Data**.

## 1 Share data with your clinic

Sofia Rossi Diabete Clinica has invited you to share your data through Dexcom CLARITY®. Dexcom CLARITY® helps you understand your glucose patterns, analyze your data and share this information with your healthcare professionals.

Your Temporary Authorization Key\*

ZNGWPCLZYLLK

- 2 Get started at: <https://clarity.dexcom.eu/share>

\*Expires: 27 October 2016

## Share Data with a New Clinic

Share your data with your healthcare clinic so that your healthcare professionals can access your glucose data through their clinic's Dexcom CLARITY® accounts. You will also have access to data uploaded through these clinic accounts. You can stop sharing at any time through the Settings page.

Please enter your Temporary Authorization Key ?

5

Please verify your date of birth:

6

7

## Share Data with a New Clinic

Please confirm your healthcare clinic's name below.


 Sophia Rossi Diabete Clinica

- 8  I consent to share my data with my clinic.

9

To stop sharing with a clinic:

- 1 Log in to CLARITY at [www.clarity.dexcom.com](http://www.clarity.dexcom.com) with your internet browser.
- 2 Click **Settings** at the top of the page.
- 3 Click **Stop Sharing with this Clinic** at the bottom of the page.

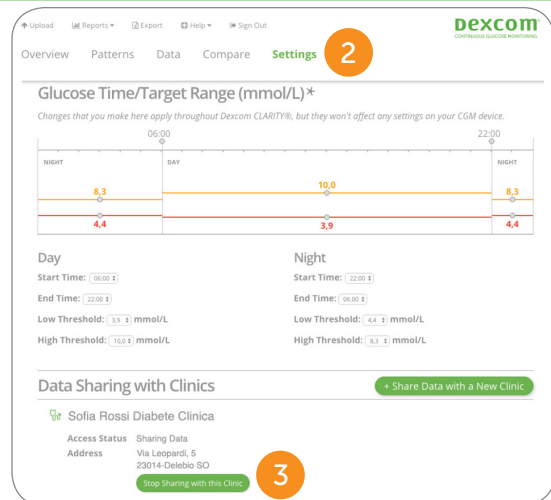

The screenshot shows the 'Settings' page in the Dexcom CLARITY web interface. At the top, there are navigation tabs: Overview, Patterns, Data, Compare, and Settings (highlighted with a '2'). Below the tabs is a 'Glucose Time/Target Range (mmol/L)\*' section with a graph showing glucose levels over a 24-hour period. The graph has a yellow line for glucose and a red line for target range. Below the graph are settings for Day and Night, including Start Time, End Time, Low Threshold, and High Threshold. At the bottom, there is a 'Data Sharing with Clinics' section with a green button '+ Share Data with a New Clinic'. Below this, there is a list of clinics, with 'Sofia Rossi Diabete Clinica' selected. Underneath the clinic name, there are fields for Access Status, Sharing Data, and Address. A green button 'Stop Sharing with this Clinic' (highlighted with a '3') is located at the bottom of the clinic entry.

\*Change your language and unit of measure preferences on the Settings page.



Dexcom  
CLARITY®

#### BRIEF SAFETY STATEMENT

The web-based Dexcom CLARITY® software is intended for use by both home users and healthcare professionals to assist people with diabetes in the review, analysis and evaluation of historical CGM data to support effective diabetes management. It is intended for use as an accessory to CGM devices with data interface capabilities. The software should not be relied on for medical advice. Home users must consult a healthcare professional before making any medical interpretation or therapy adjustments from the information in these reports. Healthcare professionals should use information from the software in conjunction with other clinical information available to them.

©2016 Dexcom Inc. All rights reserved. Dexcom CLARITY is not yet available for all countries. See [www.clarity.dexcom.eu](http://www.clarity.dexcom.eu) for details.

LBL014340 Rev 001 MT24268 PR-100-120