

## Instructions for Downloading a Dexcom Receiver into Dexcom CLARITY

1. Log into the Dexcom CLARITY account at: [www.clarity.dexcom.eu/professional/](http://www.clarity.dexcom.eu/professional/)  
(If your clinic does not yet have a CLARITY clinic account you will need to create one prior to step one – Follow the steps on screen by clicking on **Register Now**).
2. Select **Add New Patient**.
3. Enter patient details (name and DOB) and select **Save**.
4. Select **Upload Data**.  
(If this is the first time a receiver has been uploaded to Dexcom CLARITY you will be prompted to download the Dexcom CLARITY Uploader (it will take a few minutes for the download to complete). You should close any other Dexcom CLARITY text boxes that may appear on the screen).
5. Plug the Dexcom Receiver into the USB port of the computer.
6. Click **Upload** on your computer screen.
7. Upload will occur.
8. Once upload is complete, select **Go to Interactive Report**.
9. Remove Receiver.
10. At the top of the screen click on the **Date Range** and select the **Start Date** and **End Date** patient used the Dexcom Receiver for.
11. Click on **Download Report** and select the report you would like to export to a PDF document and save.

**NOTE: Always create the patient from inside the clinic account. DO NOT link the Dexcom Receiver to a personal Dexcom CLARITY account (this can never be unpaired and leads to direct uploading into a specific personal account).**