


diasend® Personal Uploader for t:slim X2™

Getting Started Guide



1 Log in or start a new account at www.diasend.com.au

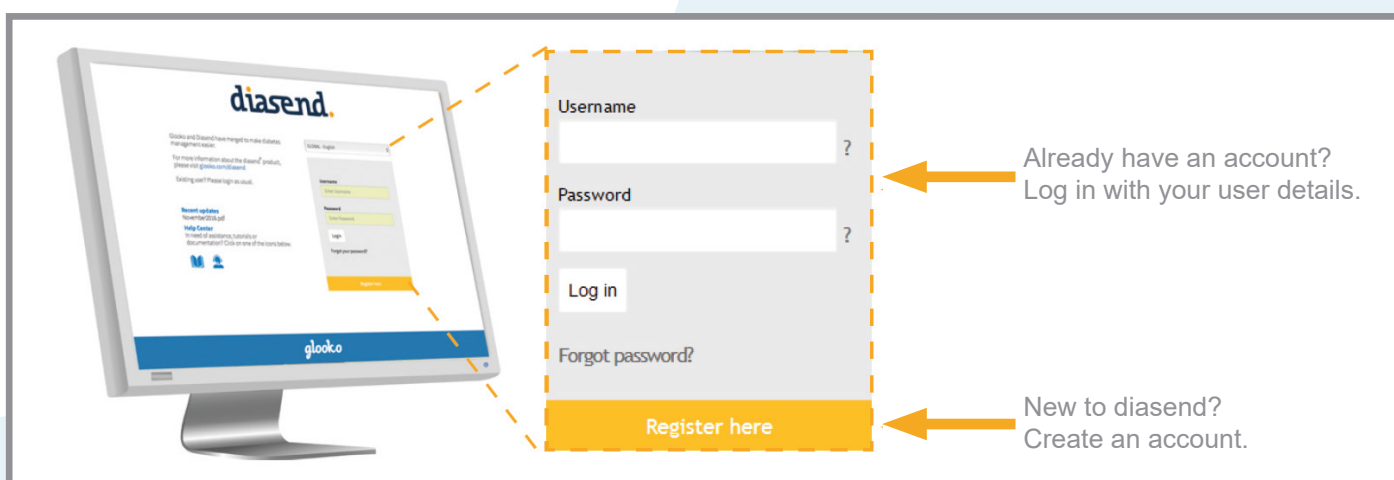


Note: Choose the icon for your country of residence.


2 Download the diasend Uploader software



3 Save the installation file



You have chosen to open:

 **DiasendUploaderPatient2.4.0_BuildR2e02.exe**


Which is a: binary file (16,2 MB)
From: <https://www.diasend.com>

Would you like to save this file?

Save File **Cancel**

*Note: Compatible operating systems:
Windows 7, 8 and 10, macOS X 10.6-10.13
Visit <https://goo.gl/YdGju1> to stay up to date with any compatibility changes.*

4 Locate the installation file and run the file

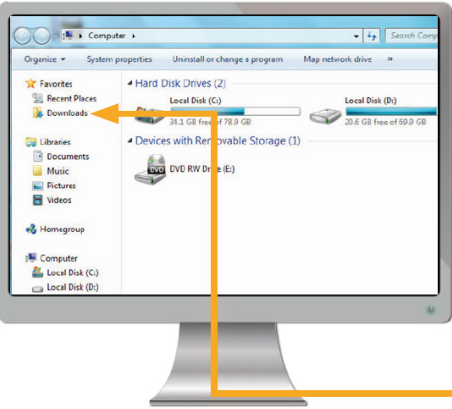


Depending on your web browser, you may need to search for the downloaded installation file, or you may just get a pop up window to run the file.

If you are a PC user: look in your recent downloads folder.

If you are a Mac user: look in the downloads folder in Finder.


5 Start diasend Uploader




Double click on the diasend Uploader icon on your desktop to start the program.

Note: Make sure you have restarted your computer before starting up diasend Uploader for the first time.

6 Connect your diabetes device cable to the computer




Connect the microUSB cable to the computer. Once the cable is connected, wait a few seconds for the cable drivers to be recognised, then connect to the t:slim X2 Insulin Pump.



Note: Make sure to use the cable that came with your t:slim X2 Insulin Pump.

7 Log in at www.diasend.com to view your data

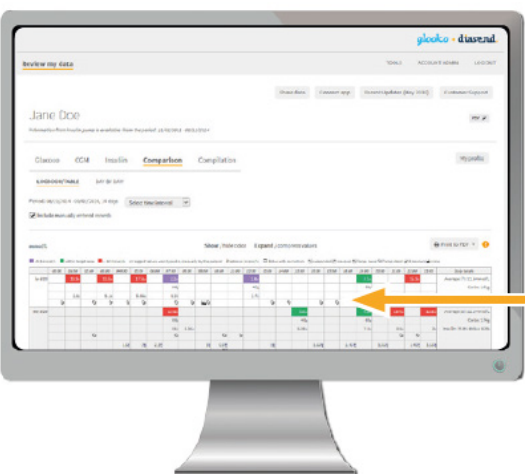


All your uploaded data is presented in easy-to-read tables and graphs.

The reports are divided into the following tabs:

Glucose CGM Insulin Comparison Compilation


8 Share your data with a clinic

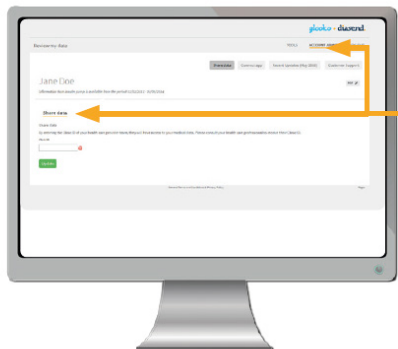

You can share your data with your healthcare provider by entering their clinic ID, under the tabs **Account admin** and **Share data**. Don't forget to click **Update**.

ACCOUNT ADMIN

Share data

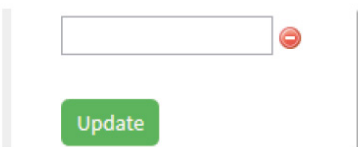

9 View your uploaded data in the diasend Mobile app



Download the app from Google Play or the App Store.
Log in with your personal diasend account details or create a new account.

With the diasend Mobile app you get:

- A summary of the latest uploaded data
- Easy to grasp reports on your insulin intake and glucose levels over time

For more information or for assistance, please contact us on
1300 851 056 or diabetes@amsl.com.au

amsldiabetes.com.au

