

Infusion Sets & Cartridges for the t:slim X2™ Insulin Pump



As of the 1st of February 2019, there will be new proprietary t:slim X2 insulin pump infusion sets and cartridges distributed by your local NDSS Access point/pharmacy. These new infusion sets and cartridges are part of the new **t:lock range** which are compatible with the t:slim X2 insulin pump.

What does this mean?

This means users will need to transition to use the new t:lock cartridge and infusion sets from the 1st of February, 2019. t:slim X2 insulin pump Luer-Lock cartridges will remain available to order during the transitional period, ending on the 1st of September, 2019.

What is the difference between the Tandem t:lock Cartridge and the Tandem Luer-Lock Cartridge?

The new t:lock cartridge was designed based on direct customer feedback related to the process of filling infusion sets with insulin. The new design:







Reduces the time required to fill infusion sets by 30 seconds



Lowers the amount of insulin used in the fill process by 4.4 units



t:lock Infusion Sets and Cartridge compatibility

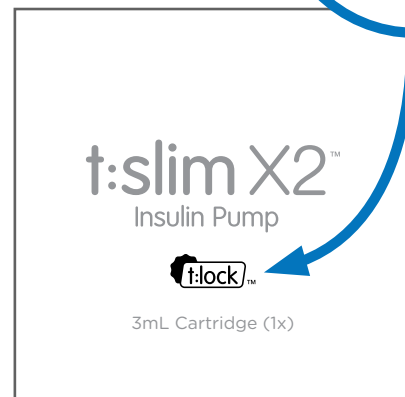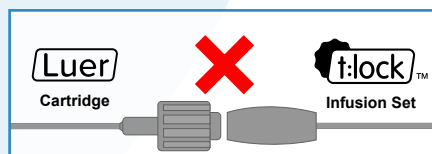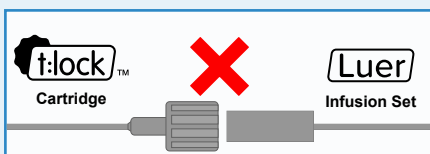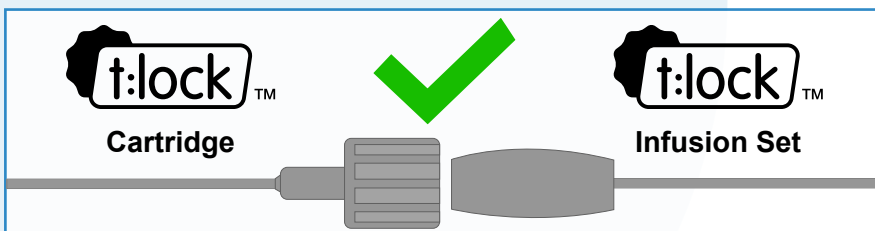

Introducing the new t:lock range

All of the below infusion sets are made by the company Unomedical, which means that there will be a compatible t:lock infusion set and cartridge corresponding to the current Luer-Lock infusion sets you are using.

Please review the table below to help you match what you are currently using and what you can switch over to. These new t:lock infusion sets will only work with the Tandem t:lock cartridge.

Current Luer-Lock Infusion Sets		t:lock Equivalent Range	
Animas Inset™ II		Tandem AutoSoft™ 90 t:lock	
Animas Inset™ 30		Tandem AutoSoft™ 30 t:lock	
Animas Comfort™ and Comfort™ Short		Tandem VariSoft™ t:lock	
Animas Contact™ Detach		Tandem TruSteel™ t:lock	
Current Cartridge	NDSS Code	t:lock Equivalent Cartridge	NDSS Code
Tandem Luer-Lock Cartridge	720	Tandem t:lock Cartridge	722

Ordering the new t:lock Infusion Sets and Cartridges

For those currently using the Inset II, Inset 30, Comfort Short or Contact Detach infusion sets alongside the Tandem Luer-Lock cartridge, these can still be used until stock is depleted. Once a new order needs to be placed for the infusion sets, a switch should be made to the new t:lock compatible infusion set and cartridge range.

From the 1st of February 2019, users will be able to order their t:lock supplies through their local pharmacy/NDSS Access Point. t:slim X2 insulin pump Luer-Lock cartridges will remain available during the transitional period, ending on the **1st of September, 2019**.

For more information on the type of infusion sets available and relevant NDSS codes, please visit www.ndss.com.au/product-and-supply

Always read the label and use only as directed. Read the warnings available on www.amsl diabetes.com.au/resources before purchasing. Consult your healthcare professional to see which products are right for you.

If you require any assistance with this transition to the new t:lock infusion sets and cartridges range, please contact the AMSL Diabetes Customer Care Team on **1300 851 056** or diabetes@amsl.com.au

amsl diabetes.com.au    

ARTG 116234, PR-100-287 August 2019

