



Diasend PDF Wizard Guide


diasend[®]
by glooko

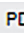

Diasend makes it simple to download and consolidate your patient's diabetes management data in the form of a PDF. For an extensive download of multiple charts, graphs and data, click on the PDF Wizard button  in the top right corner and follow the instructions below.

- 1 Set the end date of the desired time period for the data you wish to obtain.
- 2 Select as many or as few reports of data to download. The PDF Wizard allows you to consolidate a range of data including glucose, insulin and CGM reports.
- 3 Select the desired time interval for each data set you have chosen. You may set independent time periods for each data set chosen.
- 4 When you are finished selecting your data and time intervals, scroll to the bottom of the window and click the Create Report button.
- 5 You can also click on the PDF icon in the Patient List to create your compiled PDF report or click on the downward pointing arrow to select the pre-set profile you wish to view/print.
- 6 The PDF Wizard can also be found under Settings in your account. There you can also add new customised report profiles.

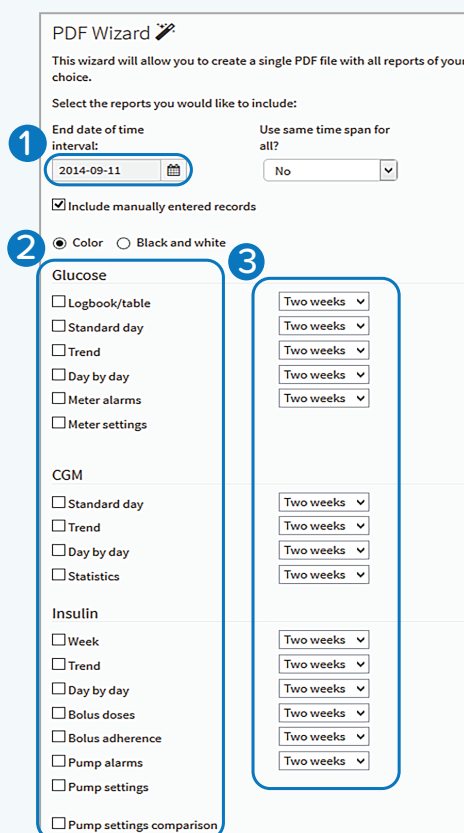

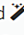PDF Wizard 

This wizard will allow you to create a single PDF file with all reports of your choice.

Select the reports you would like to include:

End date of time interval: Use same time span for all? ☐ No ☐ Yes

☒ Include manually entered records

☒ Color ☐ Black and white

Glucose

- ☐ Logbook/table
- ☐ Standard day
- ☐ Trend
- ☐ Day by day
- ☐ Meter alarms
- ☐ Meter settings

CGM



- ☐ Standard day
- ☐ Trend
- ☐ Day by day
- ☐ Statistics

Insulin

- ☐ Week
- ☐ Trend
- ☐ Day by day
- ☐ Bolus doses
- ☐ Bolus adherence
- ☐ Pump alarms
- ☐ Pump settings
- ☐ Pump settings comparison

Time intervals for each data set (all set to 'Two weeks'):

- Glucose: Two weeks, Two weeks, Two weeks, Two weeks
- CGM: Two weeks, Two weeks, Two weeks
- Insulin: Two weeks, Two weeks, Two weeks, Two weeks, Two weeks, Two weeks

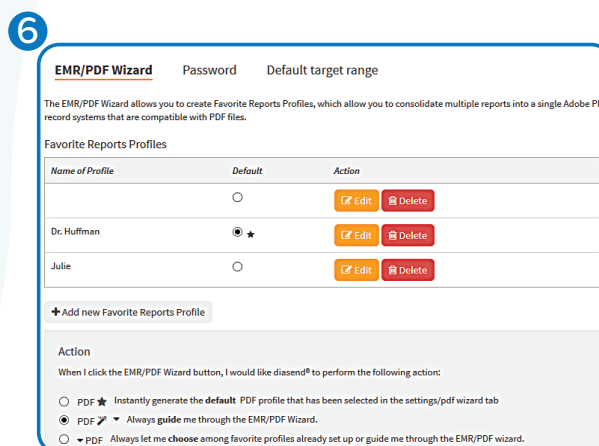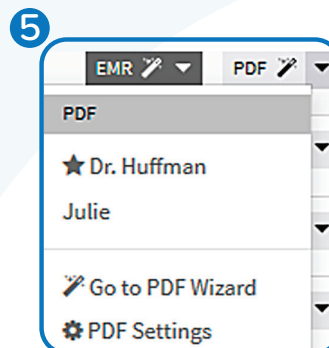

EMR/PDF Wizard Password Default target range

The EMR/PDF Wizard allows you to create Favorite Reports Profiles, which allow you to consolidate multiple reports into a single Adobe PDF record systems that are compatible with PDF files.


Favorite Reports Profiles

Name of Profile	Default	Action
	<input type="radio"/>	<input checked="" type="checkbox"/> Edit <input type="checkbox"/> Delete
Dr. Huffman	<input checked="" type="radio"/> *	<input checked="" type="checkbox"/> Edit <input type="checkbox"/> Delete
Julie	<input type="radio"/>	<input checked="" type="checkbox"/> Edit <input type="checkbox"/> Delete

+ Add new Favorite Reports Profile

Action

When I click the EMR/PDF Wizard button, I would like diasend® to perform the following action:

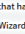- ☐ PDF ★ Instantly generate the default PDF profile that has been selected in the settings/pdf wizard tab
- ☒ PDF  Always guide me through the EMR/PDF Wizard.
- ☐ PDF Always let me choose among favorite profiles already set up or guide me through the EMR/PDF wizard.

For more information on diasend by Glooko please contact us on
1300 851 056 or at diabetes@amsl.com.au

amsldiabetes.com.au



To contact Glooko directly, visit glooko.com/int or email support@diasend.com. PR-100-370 May 2020